QUICK GUIDE

AIRFLOW[®] ONE

"I FEEL GOOD"

EMS 
MAKE ME SMILE.

DOWNLOAD ON MYEMS.DENTAL

General Instructions:
installation, cleaning,
reprocessing, maintenance ...

Treatment Recommendations
Guided Biofilm Therapy

EMS Dental
Installation & Maintenance

SUMMARY

EN ENGLISH ————— p.4

FR FRANÇAIS ————— p.8

WELCOME

We thank you for your trust in our newest product, the AIRFLOW® ONE, which will guide you into the world of GBT - GUIDED BIOFILM THERAPY.

Register your new unit on www.myems.dental and get an extra year of warranty. This warranty is only valid if you have completed your first yearly maintenance within 12 months after purchase.


Before installation, download & read the instructions for use (www.myems.dental).


Connections

- + **EH-142-US** Air hose to the cabinet / dental unit.
- + **EG-110-US** Water hose to the cabinet / dental unit.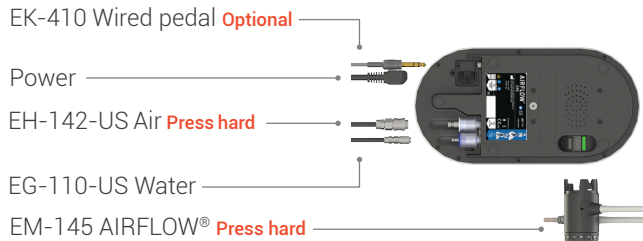

Ensure the cords are correctly connected.

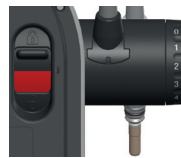Install the batteries in the pedal.

PLUS Powder: Sub and Supragingival.

Insert the powder chamber.

Functions


Powder chamber pressurization

ON/OFF

Power set

NORMAL Press gently

BOOST Press hard

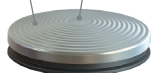CLASSIC Powder: Supragingival on enamel only.

The use of the WATER Bottle is optional.

If the bottle is not attached, the device will use the external water supply.

The bluetooth communication is compliant with the Federal Communications Commission (FCC) rules and regulations regarding unlicensed transmissions "FCC 47 CFR Part 15, Subpart B".

CLEAN

DISINFECTION of the irrigation lines

! Consult the instruction for use (FB-621).

Refer to the chapter of water line cleaning & disinfection of your AIRFLOW® ONE.

CLEANING AND REPROCESSING between each patient

Clean the external surface of the device with Hu-FriedyAdvantaClear™ Surface Disinfectant or other compatible intermediate-level surface disinfectant available in your country.

Reprocess your handpieces

MAINTENANCE

EN

INSPECT

Check the blue water filter and change it if needed.

ANNUAL Preventive maintenance

Send your device to the EMS repair center.


TROUBLESHOOTING


Handpiece clogged

Use Easy Clean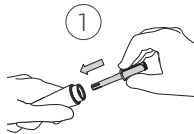

Place the Easy Clean into the handpiece.

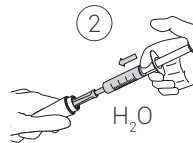
Empty water-filled syringe into the Easy Clean

Dry both sides of the handpiece with compressed air.

Service needed
Stop using your device.

Batteries low
Change pedal batteries.

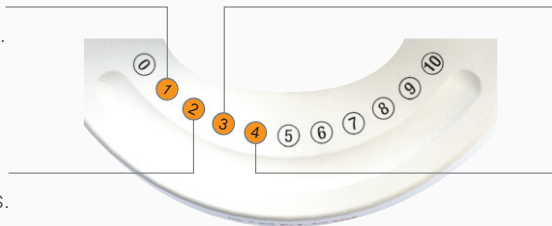Missing cord
Check your cord connection.

Overheating
Wait for cooling.
If error repeats, call service .

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AU

SYDNEY, AUSTRALIA
E.M.S. ELECTRO MEDICAL SYSTEMS OCEANIA
 Unit 1, Building 4 – Sydney Corporate Park
 160 Bourke Road
 AU - Alexandria, NSW 2015
 T. +61 405 095 867
 E-mail: info@ems-oceania.com


JP

TOKYO, JAPAN
E.M.S. JAPAN BRANCH OFFICE
 Eminent Ningyoucho Bld 6F,
 9-18, Kobuna-cho, Nihombashi,
 JP - Chuo-ku, Tokyo, 103-0024
 T. +81 (0) 3-6810-9351
 F. +81 (0) 3-6810-9352
 E-mail: emsjapan@ems-ch.com

CN

SHANGHAI, CHINA
医迈斯电子医疗系统贸易(上海)有限公司
E.M.S. ELECTRO MEDICAL SYSTEMS TRADING (SHANGHAI) CO., LTD.
 24A, Jin Sui Mansion,
 N°379 Pudong Nan Rd.
 CN - Shanghai 200120
 Tel. +86 21 3363 2323
 E-mail: emschina@ems-ch.com

FOLLOW US

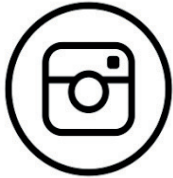EMS Dental

EMS Dental -
International


EMS Dental

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ONLY USE ORIGINAL AIRFLOW® POWDERS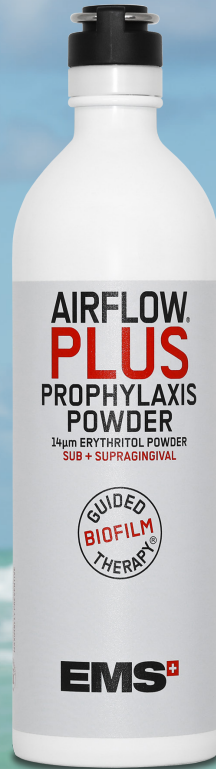

Beware of so-called „EMS compatible“ powders in the market, which can damage oral tissues and the device. See here a nozzle completely damaged by such aggressive powders.

- ▶ Use only EMS powders with your EMS devices.
- ▶ The use of not authorized powders by EMS will result in a loss of warranty.

CE 0124

EMS 
PRINTED IN SWITZERLAND
WWW.EMS-DENTAL.COM